## PROTÉSICAS

---

### Dos opciones:

- Lens 55 Prosthetic

(4 colores no modificables)

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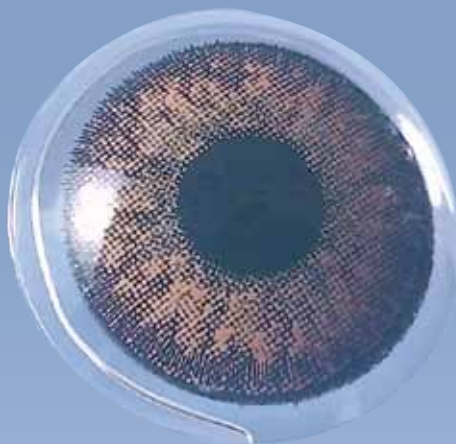
- Lens 55 Prosthetic

(4 unmodifiable colors)

- Prosthetic Service is not a contact lens. It is a Lab Service.

Prosthetic Service over K-38 Rx

(K-38 Rx clear lens is certificated and after fitting Prosthetic Service is done on the same K 38 Rx )

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## PROSTHETIC SERVICE

The prosthetic service is not a contact lens. It is a laboratory service on a K-38 Rx contact lens fitted. (Cleaning system: We recommend the use of peroxide solution with catalyst ring pag 72 Hydrogen Peroxide with ring).

Fitting a Prosthetic Lens:

- 1.- Lens Selection
- 2.- Type of Prosthetics Selection
- 3.- Color and Shade Selection
- 4.- Complete Prosthetic Lens Order Form

- LENS SELECTION

From keratometry (take measurement of the good eye if it is not possible from the damaged one), select the base curve and diameter. Fit a clear lens to ensure this offers wearing comfort, stability, mobility, and lens wearing (Make sure the lens is comfortable after 8 hours of contact lens wear).

**NOTE:** Once the lens is tinted, changes in lens parameters are not allowed.

In addition to the data obtained from the keratometry process, return the lens to be tinted together with close-up photographs of both eyes in the same picture (preferably in jpg format). Photographs should be taken with natural light,

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and clearly show the color and shade.

Servilens may apply the tinting process in different (38% & 55% water content) lens materials. Since equally good prosthetic results can be achieved with all these materials, we recommend to use yearly replacement lens materials (with 38% content water) for a longer durability.

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- Potencias: +25.00 a -30.00 en pasos de 0.25D.

## PROSTHETIC SERVICE

Our quality control system does not allow us to use trial lenses in the prosthetic service.

-K-38 Rx (pHema 38% H2O). New sterile lens with CE marking.

- Diameters: 13.00 - 13.50 - 14.00 - 14.50

- Base curves: 8.00 to 9.00 in 0.20D steps

- Powers: +25.00 to -30.00 in 0.25D steps.

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### Lente Tipo A

- Iris de color

Ø de Iris:

10.50 mm - 11.00 mm - 11.50 mm -  
12.00 mm - 12.50 mm

Ø pupila negra:

3.00 mm - 4.00 mm - 5.00 mm - 6.00  
mm - 7.00 mm

Lente de contacto utilizada para leucoma total o diplopia.

## SELECTING TYPE OF PROSTHETIC

### Lens Type A

- Colored iris

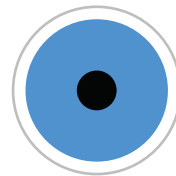
Iris Ø

10.50 mm - 11.00 mm - 11.50 mm -  
12.00 mm - 12.50 mm

Ø black pupil:

3.00 mm - 4.00 mm - 5.00 mm -  
6.00 mm - 7.00 mm

Contact lens for total leucoma or diplopia

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Ø black pupil:

3.00 mm - 4.00 mm - 5.00 mm -  
6.00 mm - 7.00 mm

Contact lens for central leucoma, or congenital cataracts.

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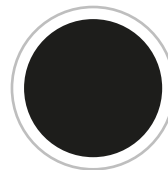
-- Black iris opaque

Iris diameter: 10.5 mm - 11.00 mm - 11.5 mm - 12.00 mm - 12.5 mm

- Occluder lens for anisometry, diplopia and amblyopia.

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## PROSTHETIC SERVICE

### 3- Color selection

Matching the patient's natural eye color to a prosthetic lens is a complex task since the eye color varies depending on many factors, especially on the lighting conditions. Please take this into consideration when photographing the iris color. Send photographs of both, the damaged eye and the eye to be matched. It is also helpful if you choose the most similar color from our color chart.

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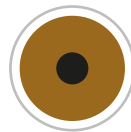
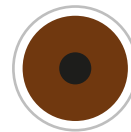
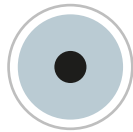
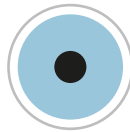
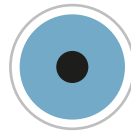
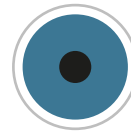
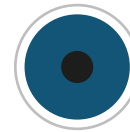
## PROSTHETIC SERVICE

### Maintenance

Do not use solutions containing bleaching agents. The best cleaning method is the hydrogen peroxide cleaner with ring, distributed by Servilens. If other cleaning systems are used, we do not guarantee the color stability. Otherwise, the lens will maintain its color one year. The lens is immersed in the peroxide solution for only 20 minutes.

### Color Retouching

Warranty allows the lens be tinted again within 40 days.

### Complete prosthetic lens order form

For ordering a prosthetic lens, please send the order form duly completed, and a photograph of both eyes.

**WEB**  
**lens55.com**