INSTRUCCIONES DE USO 47844 Rev(5-4) 2017-12

Lentes de Contacto

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Fit and Cover Servilens, Avd. Ilustración, 112 - 18016 Granada - Spain

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Information leaflet, to be provided to wearers of contact lenses by their contact lens specialist/optometrist

Patient Instructions 47844

GB

FOLLOW THE INSTRUCTIONS OF YOUR EYE CARE PROFESSIONAL, STRICTLY PLEASE READ CAREFULLY AND KEEP THIS INFORMATION FOR FUTURE USE. Contact lenses should always be fitted by an eye care professional, who will recommend the wearing and replacement schedule best suited for your needs.

These lenses are intended for vision correction and 5% (*) percentage absorption of blue light and ultraviolet light (UVA 84.2%, 97.6% UVB) which induces retinal damage as AMD(1). Lens 55 CSR contact lenses are specially recommended when the eye is exposed to light sources containing blue light as sunlight or artificial light sources such as LED, computers screens, PC tablets or Smartphone, (2-3) always in people with healthy eyes and under the supervision of an eye care professional.

These contact lenses are suitable for driving and do not replace the use of sunglasses. INSTRUCTIONS FOR HANDLING YOUR CONTACT LENSES. See in figures 1 . 5. Do not use contact lenses if pack is opened or damaged. Do not use the contact lens if the lens is damaged. If the lenses dry out. Re-wet with Disinfecting, soaking or rinsing solution then ask your eye care professional. Your eye care professional should provide you with detailed contact lenses insertion and removal instructions, as well as information on the safe use of your lenses. Wear the contact lenses during the time indicated by your eye care professional. Check your vision periodically with your eye care professional and follow the instructions use. Blister: gently shake the blister pack before opening. Peel back the foil lid. Pour the lens into the palm of your hand or if necessary, gently slide the lens out of the container, or open as directed by your eye care professional. CONTACT LENSES MAINTENANCE- IT IS ESSENTIAL THE CORRECT CARE OF YOUR CONTACT LENSES- Your eye care professional should choose the lens care system most suitable for you. It is essential that you follow the labelling instructions on your lens care products- Do not use heat disinfection nor any product designed for hard or rigid gas permeable lenses. NEVER allow contact lenses to come in contact with non-sterile water (including tap water) as microbial contamination can occur, which may lead to permanent eye damage. Do not change the contact lens maintenance without contacting with your eye care professional. Ask your eye care professional about wearing contact lenses during sporting activities. Remove lenses when exposed to noxious or irritating vapour. IMPORTANT PRECAUTIONS When you are wearing contact lenses, check your eyes every day. Your eyes should look well, you should feel comfortable with them and your vision should be clear. If it is not, remove your lenses immediately and contact your eye care professional. Unusual symptoms, if ignored, can lead to serious complications. Do not use contact lenses in: corneal dystrophies, corneal ulcers, inflammation of cornea or the eye, allergic eye conditions. Do not share your contact lens with anyone else. Do not wear your lenses when using eye drops (unless advised to do so by a medical practitioner). Do not use saliva to wet your lenses. Daily USE. FOR ONLY USE DURING THE DAY (Do not use contact lenses while you are sleeping). No special conditions are required for its disposal.

RECOMMENDED REPLACEMENT FOR CONTACT LENSES And Materials

Lens 55 CSR / Toric (30 days) (Filcon IV 1), Contact lens in 0.9 % buffered saline solution)

Sterility: lenses are sterilised by steam (sterility is only guaranteed in the closed and undamaged sealed package). Do not use the lenses if the packaging has been tampered with.

- (1) Sparrow, J.R., A.S. Miller, and J. Zhou, Blue light-absorbing intraocular lens and retinal pigment epithelium protection in vitro. J Cataract Refract Surg, 2004. 30(4): p. 873-8.
 - (2) Behar-Cohen F, Martinsons C, Vienot F, Zissis G, Barlier-Salsi A, Cesarini JP, Enouf O, Garcia M, Picaud S, Attia D: Light-emitting diodes (LED) for domestic lighting: any risks for the eye? Prog Retin Eye Res 2011, 30(4):239-257.
 - (3) Chamorro, E., et al., Effects of Light-emitting Diode Radiations on Human Retinal Pigment Epithelial Cells In Vitro. Photochem Photobiol, 2013. 89(2): p. 468-73
- (*) Measurements according to standards EN 1836 - EN 18369-2.

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